Installation instructions

Montageanweisung

ACROBAT
Swing

Wall and ceiling models
Wand-und Deckenausführung

Table of contents Page

1	Static evidence	8
2	Selection of fastening agent	9
3	Wall model	10
3.1	Installing the wall bearing	10
3.2	Installing the extension arm with spring arm	11
4	Ceiling model	12
4.1	Installing the ceiling plate	12
4.2	Shortening the spacer tube	13
4.3	Mounting the extension arm with spring arm ...	14
5	Mounting the end device	15
6	Setting the spring force	17
7	Technical Data	18
8	Drilling template	19

Packing list, extension arm and spring arm

- 1 extension and spring arm (pre-mounted)

Accessories required for installation:

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Instructions for safe assembly

Dear user,

The equipment may only be assembled by a hospital technician or a comparably qualified person.

The following persons shall be considered as qualified personnel:

- Persons who underwent special professional training in the field of medicine or medical engineering,
- Persons who can assess their work and recognise the potential hazards involved on the basis of their professional experience and instruction in safety-relevant regulations.
- In States where the performance of tasks in the medical or medical engineering sector are subject to certification, qualified personnel must have obtained the corresponding certificate.

Dear installer,

Please read these installation instructions very carefully and follow the safety instructions and requirements of these installation instructions.

If there are any peculiar problems that have not been treated in sufficient detail in these installation instructions, please contact your supplier for your own safety.

Changes to the product are not permitted.

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Instructions for safe assembly

Intended purpose:

- The Acrobat appliances have been designed for the carrying and positioning of medical examination lamps and flat screens and also for the power supply of the devices. Depending on the transformer version, e.g. 13,2V, 13,5V, 14V, 24V (without transformer, 230V), 50/60Hz are available (not optionally adjustable).
- The appliance is suitable for continuous operation.
- The maximum load indicated on the rating plate of the individual appliance version must not be exceeded (see Chapter 5 "Mounting the end device").

Incorrect use

- The maximum loading capacity of the appliance and its components as specified in "Chapter 7, Technical Data" must not be exceeded.

Contraindications

- The ACROBAT Swing pendant system must not be used close to strong magnetic fields.
- No BF or CF application parts in accordance with IEC 60601-1 may be directly connected to the pendant system.

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Instructions for safe assembly

Supplementary symbols used in the safety instructions:

Electrical shock:

Warns of electric shock which can lead to severe injury or even death.

Pendant system crashing:

Warns of the risk of the pendant system suddenly crashing because the max. loading capacity has been exceeded.

Sudden release of spring arm:

Warns that the spring arm may jump up suddenly while dismantling the end device.

Graphic symbols on the appliance and/or on the packaging:

Observe the Operating Instructions:

Read these Operating Instructions carefully prior to initial operation of the pendant system. This ensures that you benefit from all the advantages of the pendant system and prevents any risk of injury or damage.

Observe the maximum loading capacity:

Warns of exceeding the maximum loading capacity (payload) on the pendant system of the adaption or the end device (e.g. flat screen, OR lamp, etc.).

CE mark:

Ondal declares that the products comply with the relevant regulations set forth in the applicable European Directives.

Recognised cUL component:

This component has been recognised by Underwriters Laboratories Inc. Representative samples of this product have been reviewed by UL and comply with the applicable requirements.

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Instructions for safe assembly

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Working range of the spring arm

When mounting the wall bearing and the ceiling mount fixture, it must be ensured that the end device can be positioned within its desired working range without the pendant system being constantly moved to the existing height or rotational stop positions during operation.

Otherwise there is a risk of the pendant system being damaged.

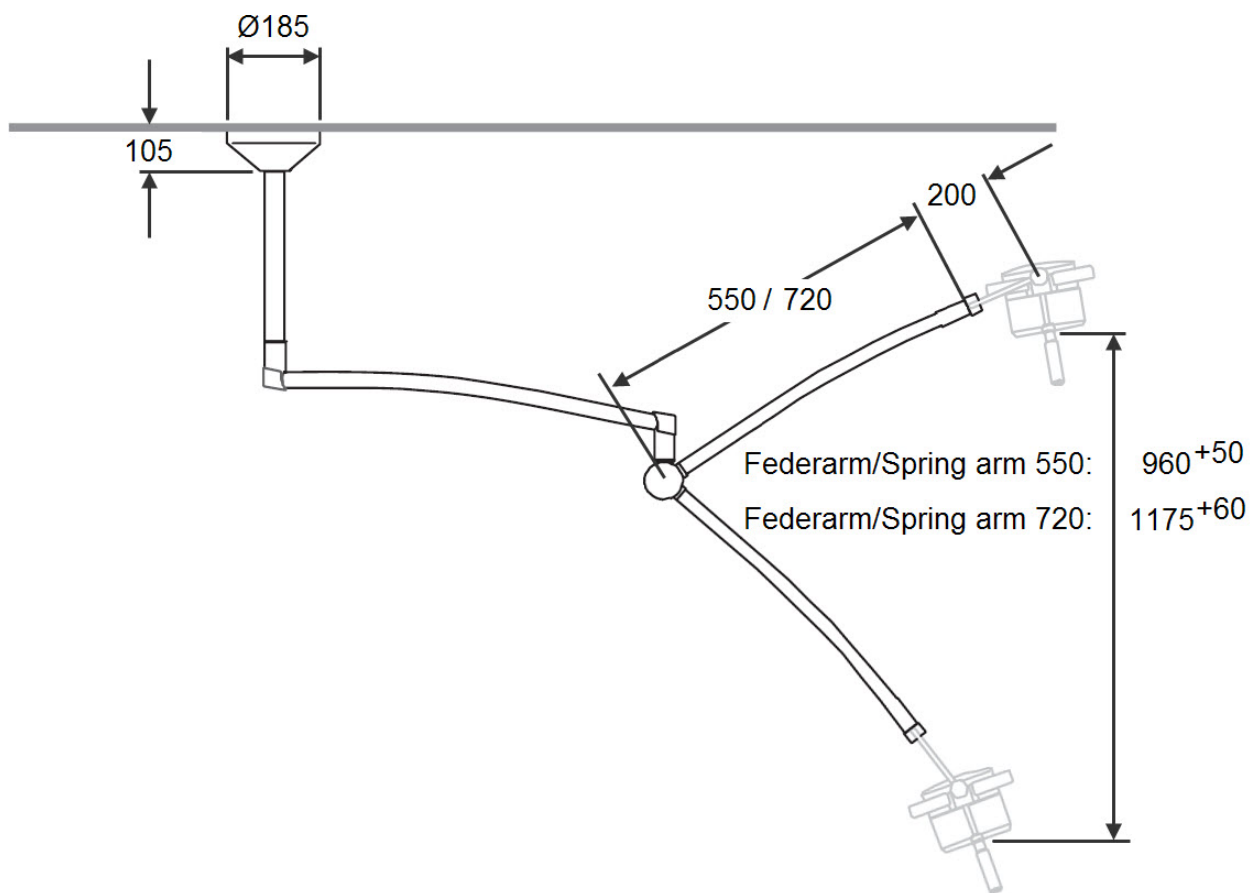
(The Figure shows the ceiling mount fixture.)

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Mounting / dismantling:

CAUTION - Electrical shock:

To prevent the risk of electric shock, the appliance must be connected to a supply network with a protective conductor.

- The appliance must be connected in such a way that it can be disconnected from the mains at all poles and at the same time.

Atmospheric pressure:

Indicates the permissible atmospheric pressure values in a range from 500 hPa to 1060 hPa for transport and storage.

Relative humidity:

Indicates the permissible humidity values in a range from 10% to 75% for transport and storage.

Ambient temperature:

Indicates the permissible ambient temperature values in a range from -25 °C to 70 °C for transport and storage.

Ambient conditions for storage and transport

The following storage conditions apply for storage times of up to 15 weeks:

Ambient temperature: -25 °C to 70 °C;

Relative humidity: 10 % to 75 %;

Atmospheric pressure: 500 hPa to 1060 hPa.

Store only in indoor rooms; after this time, the values specified for the ambient conditions for operation apply.

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Deutsch

1 Static inspection**👉 NOTE:**

The static (structural) inspection must be carried out before the installation of the wall or ceiling fastening.

- The strength of the construction must be designed, checked and certified by a structural engineer.
- The respective regional construction regulations that apply must be followed.
- If an erroneous hole is drilled by mistake, e.g. drilling of a reinforcement rod, the structural engineer who is responsible must be contacted, since adequate static load distribution in the ceiling may have been jeopardized.

Declaration of acceptance:

It is hereby certified that the supporting wall/ceiling and the anchoring of the Acrobat Swing is safe and adequately strong.

Project: _____

Anchoring (please check the one that is applicable):

- with counter-plate
- other

Location: _____

Signature/Stamp (Structural Engineer / Construction Authority)

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Deutsch

2 Selection of the fastening agent

- The selection of the fastening agent according to table 1, „Loading data“ and the safe execution of the fastening is the responsibility of the person in charge of the installation.
- In case of light construction walls, we recommend fastening with a counter-plate (not shipped).

WARNING - Load data:

- The load torque on the spring arm of the wall-mounted device must not exceed 39 Nm.
- The load torque on the spring arm of the ceiling-mounted device must not exceed 30 Nm.
- The load data indicated do not include safety factors. The specified regional safety factors must be taken into account!

The load data of the wall and ceiling models can be taken from the following table.

Table 1:

Loading data for wall mount

Load torque wall mount	Vertical force of weight	Extraction force for each dowel (total 2 pieces)
110 Nm	97 N	625 N

Loading data for ceiling mount

Load torque ceiling mount	Vertical force of weight	Extraction force for each dowel (total 4 pieces)
85 Nm	147 N	405 N

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For the mains connection of the wall model, it is necessary to have a properly grounded socket in the vicinity of the connection cable.

1. Remove the two **plastic covers (1)**.
2. Make the vertical markings according to the drilling template.
3. Drill two holes according to the specifications of the manufacturer of the fastener.
4. Insert the fastener flush with the wall.

NOTE:

Continue the installation only after the binding agent has hardened completely.

5. Insert the fastener in the two **holes (2)** and bolt the wall bearing vertically to the wall.
6. Put on both the **plastic covers (1)** and check them for secure seating.

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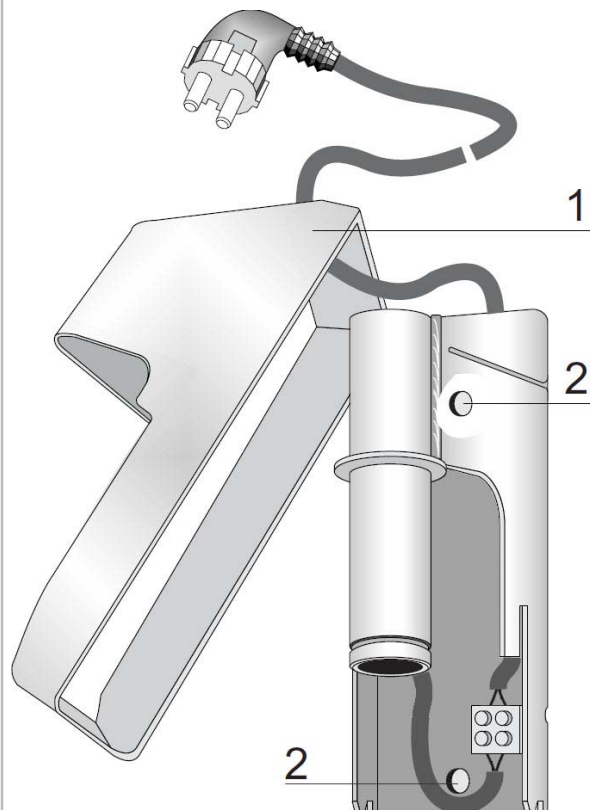
1. Unscrew the **cross-slotted screw (7)** and pull the **cover (6)** forward and remove it upwards.
2. Place the extension arm with the spring arm on the pin of the wall bearing.

⚠ CAUTION - Danger of injury

If the washer (3) and slotted cone (4) is not installed, the retaining ring (5) can get loosened. The equipment can then fall out of its mounting and cause injury. Always install the washer (3) and the slotted cone (4).

3. Install **washer (3)**.
4. Insert the **slotted cone (4)** in the **hole (1)** and install the **retaining ring (5)**.
5. Check for firm seating of the **retaining ring (5)**.
6. Set up the electrical **connection (2)**.
7. Put on the **cover (6)** and screw it tight with the **cross-slotted screw (7)**.

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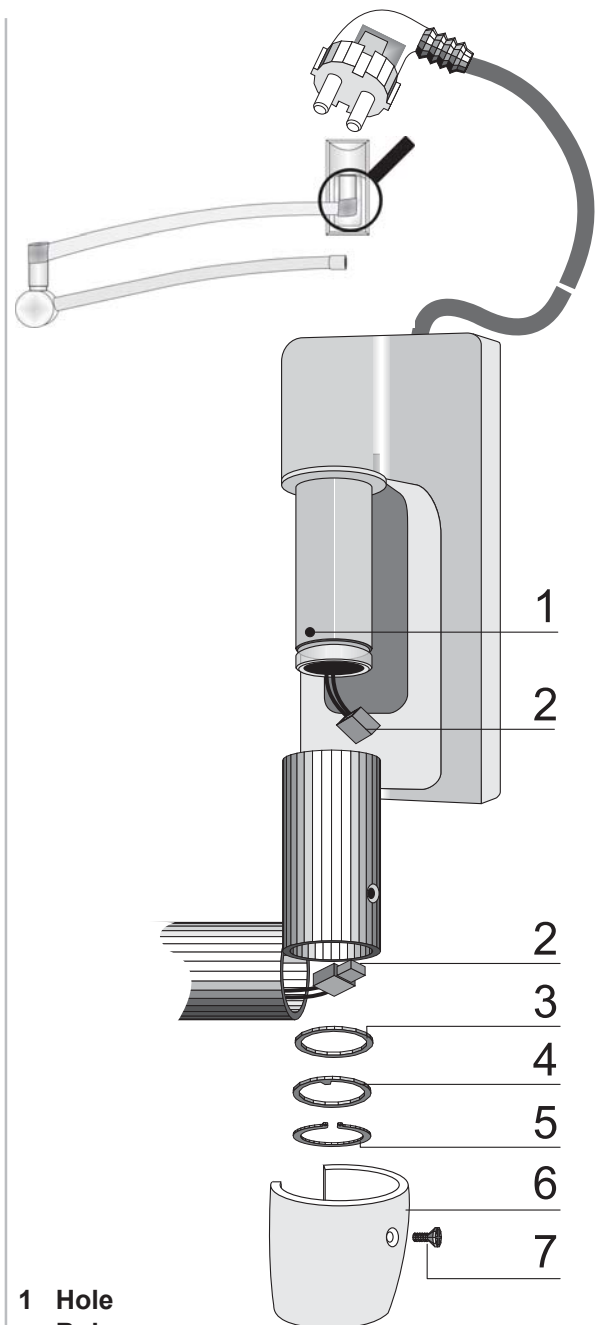
Power off the construction side supply and secure it from being switched on again.

1. Loosen the three **threaded pins (1)**, and remove the **fixture flange (2)**.
2. Mark the holes with the drilling template (on the last page).
3. Make four holes according to the specifications of the manufacturer of the fastening agent.
4. Insert the fasteners flush with the ceiling.

👉 NOTE:
Continue with the installation only after the binding agent has hardened completely.

5. Insert the fastening agent in the four **holes (4)** and screw in the **ceiling plate (3)**.
6. Align the **ceiling plate (3)** horizontally and check for firm seating.

👉 NOTE:
If the desired pass line height is not observed, the spacer tube can be shortened as described in Chapter 4.2. It is recommended to observe a pass line height of at least 2,000 mm.

7. Clamp the transformer and ground the ceiling plate.
8. Place the fixture **flange (2)** and fasten it with the three **threaded pins (1)**.

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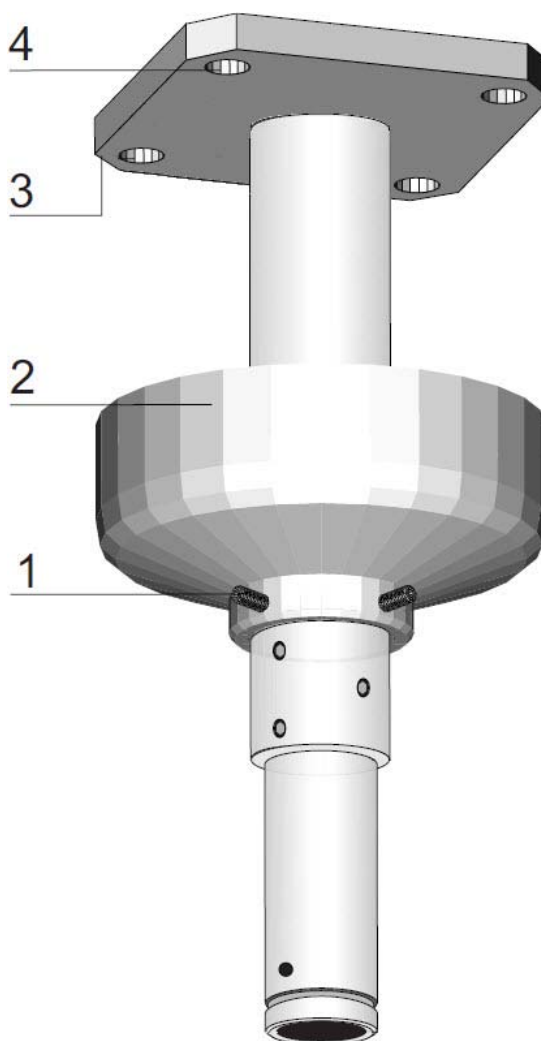
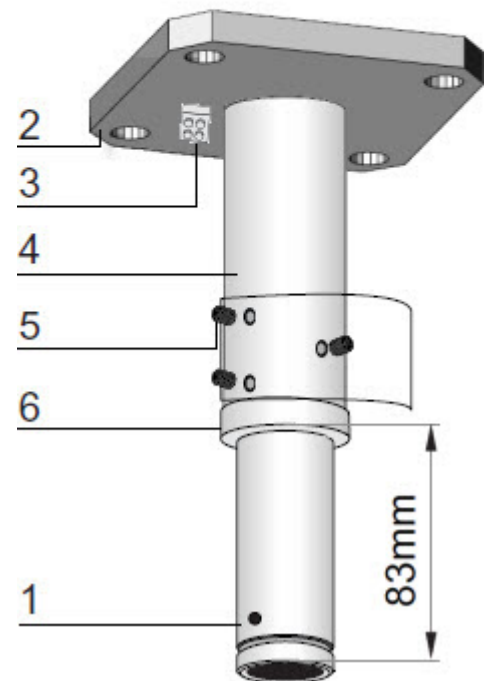
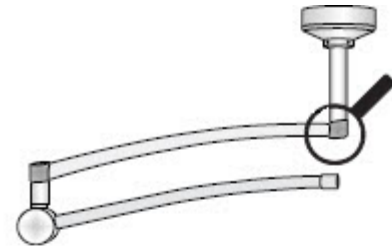
The spacer tube can be shortened by 23.5 mm, 46 mm or by a length greater than or equal to 63 mm. However, the spacer tube must be at least 110 mm long.

1. Measure the **distance X** from the lower edge of the **pin (1)** to the finished floor.
2. Unscrew the **ceiling plate (2)** from the ceiling.
3. Unclamp the **power supply cable (3)** and pull it through the **spacer tube (4)**.
3. Dismantle the **pin (1)** by unscrewing the three **threaded pins (5)**.
4. Saw off the **spacer tube (4)**, in a plani-parallel manner, according to the formula, by the **length L** and de-burr it.
5. Place the drilling template (accessory) around the lower edge of the spacer tube (4); drill any missing holes with a diameter of 5mm and cut the M6 threads. **3 threads must be available.**
6. Connect the **power supply (3)** and earth the ceiling plate.
7. Put on the accompanying **covering ring (6)**.
8. Insert the **pin (1)** in the **spacer tube (4)** in such a way that the **pin (1)** projects 83 mm. Screw on the **pin (1)** with the three threaded **pins (5)**. **3 threaded pins must be mounted!**

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1. Unscrew the **cross-slotted screw (2)**.
2. Remove the **plug (6)**.
3. Push the extender arm with the spring arm onto the pin of the spacer tube.

! CAUTION -
Danger of injury

If the washer (3) and slotted cone (4) is not installed, the retaining ring (5) can get loosened. The equipment can then fall out of its mounting and cause injury. Always install the washer (3) and the slotted cone (4).

4. Install **washer (3)**.
5. Insert the **slotted cone (4)** in the **hole (1)** and install the **retaining ring (5)**.
6. Check whether the extension arm with spring arm is seated firmly.

! WARNING -
ELECTRICAL SHOCK:
 If any electrical components (cables, plugs, etc.) are damaged, the pendant system can carry an electric voltage. Contact with energised components presents a danger to life from electric shock.

7. Plug in horizontally and push the **plug (6)** in the direction of the support and the pipe until the plug locks into position.
8. Use **cross slotted screw (2)** to install **plug (6)** carefully into position.

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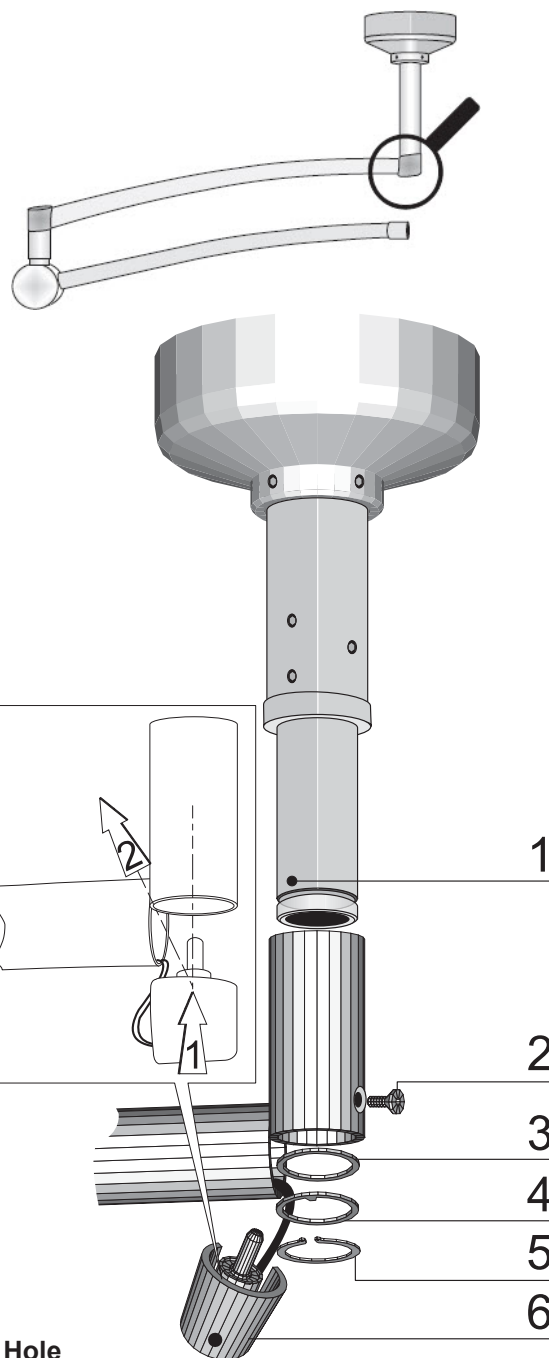
5 Mounting the end device Fig. 7-1

NOTE:

- The load torque on the spring arm of the wall-mounted device must not exceed 39 Nm.
- The load torque on the spring arm of the ceiling-mounted device must not exceed 30 Nm.

1. Power off the equipment and secure it from being switched on again.

WARNING - Danger of injury:

The spring arm, when it is pressed downward, can jump up suddenly and cause injury. Nobody should be present in the swiveling range of the spring arm during the installation of the end device.

CAUTION - Damage to the device:

The spring arm will be damaged if it is pulled beyond the lower / upper stop.

Grab the spring arm at the front end (not at the rear end) and slowly pull it downward until it reaches the stop.

2. Grab the spring arm at the front end and slowly pull it downward until it reaches the stop.
Take out the protective cap from the spring arm opening.
3. Push on the **sleeve (1)** onto the spring arm in such a way that the **slots (2)** are covered.
Continued on next page.

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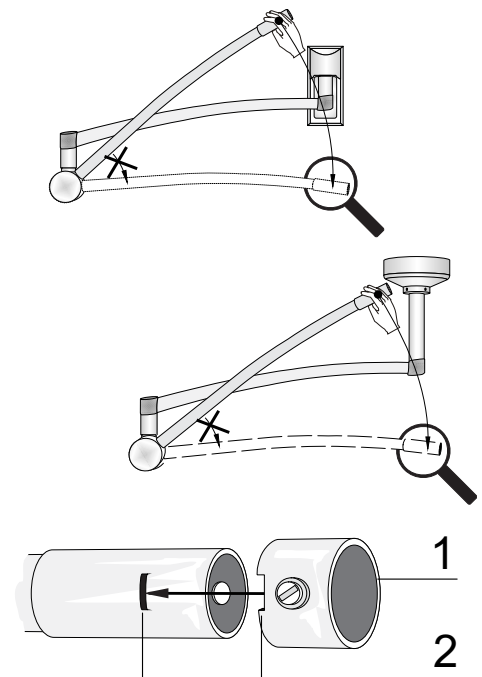
Abbildung / Figure


5 Mounting the end device Fig. 7-2

4. Push in the **end-device (3)** and the **securing segment (4)** completely in the **slot (2)**, so that the securing segment is guided in the **groove (5)**.
5. Rotate the **plastic sleeve (1)** through 180 degrees, and tighten the **braking screw (6)** till the **end-device (3)** is braked.
6. Check the secure seating of the **end-device (3)**.

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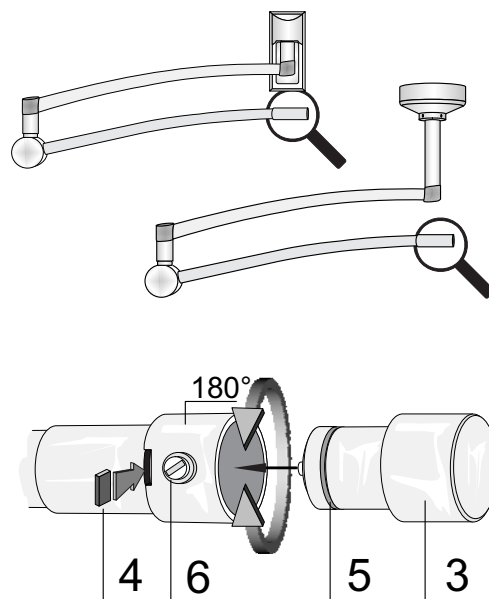6 Federkraft einstellen Abb. 8

Figure / Abbildung 8

Like every technical component, springs are subjected to natural wear. Thus, the spring force may give way and reduce after long operation, and has to be re-adjusted.

Adjust the spring force in such a way that, the spring arm with end device comes to a standstill in every desired position.

! CAUTION - Destruction of the spring arm:

The spring tension is adjusted in the upper end position.

1. Remove the left-hand joint cover half (2) from the spring arm (when looking from the end device). To do this, gently push the joint cover half out of the groove in the spring arm joint using a small slotted screwdriver.
2. Move the end device to the upper end position.
3. Insert the slotted screwdriver into the drill hole (1) and adjust the spring tension.
4. Mount the joint cover and make sure that it snaps into place.

! CAUTION - Destruction of the spring arm:

Driving in the brake screw too deeply destroys the spring arm. Tighten the brake screw with care while continually checking the braking force.

If the spring arm drops - the spring force is too low.

- The adjusting screw must be rotated to the left (in the counter-clockwise direction).

If the spring arm goes upward - the spring force is too high:

- the adjusting screw must be rotated to the right (in the clockwise direction).

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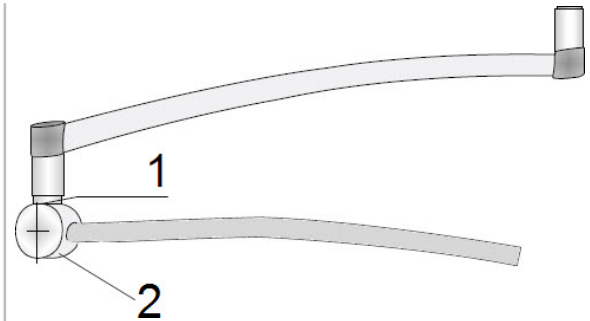
Deutsch

7 Technical Data

Electrical data	Ceiling-unit	Wall-unit	Stativ-unit
Rated voltage, depending on model	120 V / 230 V	120 V / 230 V	120 V / 230 V
Maximum power*	90 W	80 W	90 W
Nominal frequency	50 / 60 Hz	50 / 60 Hz	50 / 60 Hz
Output current at 230 V *	bis zu 10 A	bis zu 10 A	bis zu 10 A
Typ of fuses: 230 V: T 10 L 250 V prim. 115V/sec. 14V/60VA: T1,0A/250V prim. 230V/sec. 14V/60VA: T0,5A/250V prim. 103V-127V/sec. 13,2V-14V/60VA: T1,25A/250V prim. 213V-247V/sec. 13,2V-14V/60VA: T0,63A/250V prim. 230V/sec. 13,5V/60VA: T0,5A/250V prim. 115V/sec. 24V/90VA: T1,6A/250V prim. 230V/sec. 24V/90VA: T0,8A/250V			
* depending on version			
Weight in kg	Ceiling-unit	Wall-unit	Stativ-unit
Spring arm (standard)	1,8 kg	1,8 kg	1,8 kg
Extension arm	1,9 kg	1,9 kg	--
Ceiling mounting	3,0 kg - 7,9 kg	---	--
Wall bearing	---	1,8 kg	--
Stativ			3,5 kg
Stativ base			6,5 kg
Stativ base (weighted)			12,0 kg 15,0 kg 18,0 kg
Classification according to EN 60601-1			
Device of protection class I			
IP-Classification according to IEC-60529		IP20	
Suitable for continuous operation			
Classification according to guidelines 93/42/EWG, appendix IX			
Class I			
EU-Conformance			
The Acrobat Swing equipment conforms to the specifications of the guideline 93/42/EWG (medical products guideline). The Acrobat Swing equipment is only intended for connecting medical examination lamps, operation lamps and flat-screens with CE-markings and - je nach Ausführung - a maximum of 30 resp. 39 Nm. self-weight. The person operating the equipment must submit a declaration as per article 12 of guideline 93/42/EWG (medical products guidelines). Another conformance evaluation must be carried out for other end devices.			

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